

Omni Pod

To change Max bolus:

- On Home screen select Settings press select
- Select system setup, press select
- Select Bolus/Basal/Calc, press select
- Select Max Basal, press select
- Enter maximum basal rate, press enter

Determining (editing) Correction factor:

- On Home screen select settings, press select
- Select system setup, press select
- Select bolus/basal/calc, press select
- Select ratios/factors/targets, press select
- Select correction factor (to edit select time press edit and change. Press Next after each entry and when completed press done, then save)

Determining (editing) Insulin Carb Ratio:

- On Home screen select settings, press select
- Select system setup, press select
- Select bolus/basal/calc, press select
- Select ratios/factors/targets, press select
- Select IC ratio (to edit select time press edit and change. Press next after each entry and when completed press done, then save)

Determining (editing) Basal Rates:

- On Home screen select settings, press select
- Select Basal programs, press Select
- Select a program from the list, press Select
- Select View, press Select
 - **Or to Edit Basal Rate**
- Select Edit, press Select
- Select the segment to change, press edit. or choose add new, press New
- To save the newly entered segment into the basal program, press Save

Omni Pod

Setting Temporary Rates:

- On Home screen select Temp basal, press select
- Enter Temp basal rate percentage
- Enter duration of Temp basal rate, press enter
- Select Confirm to start temp basal rate

To Cancel Temp Rate:

- On Home screen select Suspend/Cancel, press select
- Select Cancel Temp rate, press select
- Select confirm to cancel temp rate.

Determining Average Insulin Use:

- On home screen select my records, press select
- Select Insulin Delivery, press select
- The current days summary is shown, use the up/down button to change to previous days.